



baby vida

oxygen monitor



instructions
& product
information





table of contents

WHAT'S INCLUDED	2
HOW IT WORKS	3
FITTING INSTRUCTIONS	4-7
APP INSTRUCTIONS	8-9
APP INDICATORS	10
CARE INSTRUCTIONS	12
PRODUCT MATERIALS	12
THE MOM BEHIND THE MONITOR	14
FAQs	16-17
WARRANTY	18-19
DISCLAIMER	20-21

what's included



OXYGEN MONITOR (INCLUDED)

The Oxygen Monitor accurately monitors your baby's oxygen level and heart rate.

**Two Support Straps and a charger are also included*



TWO SUPPORT SOCKS (INCLUDED)

Two Support Socks are included. One for 0-6 months and another for 6-12 months.



MOBILE APP (INSTRUCTIONS TO DOWNLOAD ON PAGE 9)

The App is free to download and accurately displays the information collected by the Oxygen Monitor via Bluetooth.

how it works

MONITORS AND ALARMS TO READINGS OUTSIDE OF PREDETERMINED RANGE

The Baby Vida Oxygen Monitor monitors your baby's oxygen level and heart rate. If either of these readings fall outside of a predetermined range, an alarm will sound through your smart device linked to the Baby Vida App, as long as the App Alarm is turned ON.

		PREDETERMINED RANGES	MONITOR ALARMS AT
O ₂ OXYGEN LEVEL	0-12 MONTHS	93%-100%	0%-92%
HEART RATE	0-3 MONTHS	80-180 bpm	0-79bpm or 181 bpm+
	3-12 MONTHS	70-160 bpm	0-69bpm or 161 bpm+
MANUAL SETTINGS	If you prefer, you can set your own ranges for Oxygen Level and Heart Rate. This allows you to control the levels at which the App Alarm will sound.		

**The Baby Vida Oxygen Monitor has a connection radius up to 60 feet.*



wireless



portable



easy-to-use

fitting instructions



1 With the SENSOR SIDE FACING UP, pull the Support Strap through each slit as shown. NOTE: Support Strap should be evenly distributed on each side.



2 With the sensor side still facing up, grab the RIGHT side of the Support Strap and slide through the BOTTOM LEFT slit.



3 With the sensor side still facing up, grab the LEFT side of the Support Strap and slide through the BOTTOM RIGHT slit. NOTE: Support Strap should form an X over sensor.



4 With the SENSOR SIDE FACING UP, gently place your baby's foot between the Monitor and the X of the Support Strap.

See video instructions online at www.babyvida.us

fitting instructions



5

Align the sensor to your baby's heel and connect velcro tabs on top of your baby's foot.
NOTE: Ensure power button is facing outward.



6

Gently slide the Support Sock onto your baby's foot.



7





Wrap large velcro tab to the back of your Monitor and attach smaller velcro tab to the larger. Fit should be snug.
NOTE: Additional fabric flap can be wrapped behind the Monitor, acting as a barrier between the Monitor and your baby's leg.



Ensure that the Support Sock is placed comfortably on your baby's foot and is also tight enough to hold the Oxygen Monitor in place.

See video instructions online at www.babyvida.us

app instructions

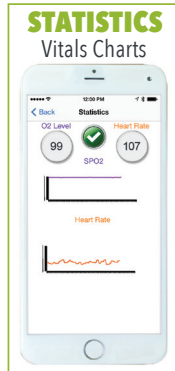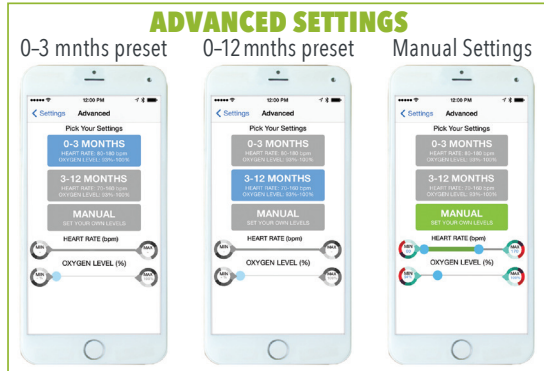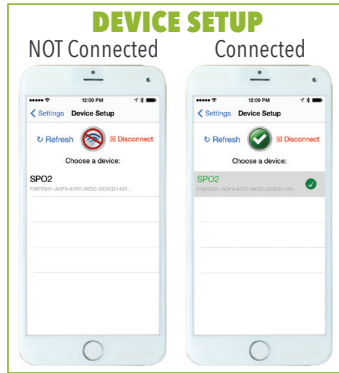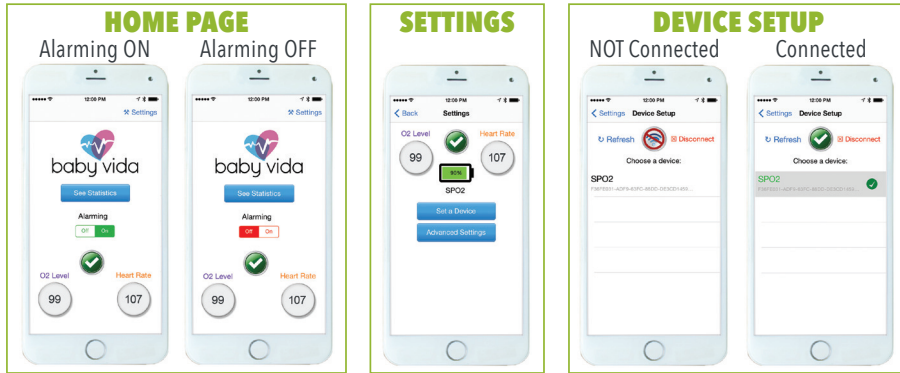

INSTALLING THE APP

- 1 On your smart device, go to the App Store
- 2 In the search field, type "Baby Vida"
- 3 When the App appears, click the FREE box in the top right to download

LINKING THE APP TO YOUR OXYGEN MONITOR

- 1 The Baby Vida Oxygen Monitor corresponds to Bluetooth. In your smart device settings, please ensure that Bluetooth is turned ON
 - 2 Open Baby Vida App
 - 3 In the top right of your screen, click "Settings"
 - 4 Your screen will say "Not Connected"
 - 5 Click "Set a Device"
 - 6 Under "Choose a Device" you will see "SP02-" followed by your monitor's serial number. **If you do not see your device listed, please make sure the Oxygen Monitor is powered ON by pushing the power button on the Monitor itself. You will know it is ON when the sensor glows red.*
 - 7 Touch that serial number (it will change to green)
 - 8 The top right of your screen should now read "Connected"
 - 9 Click "<Settings" and then "Advanced Settings"
 - 10 In "Advanced Settings" select your baby's age (preset ranges) or manually set your own ranges
 - 11 Click "<Back" to return to the home screen and choose to have alarming on or off
 - 12 Your Baby Vida Oxygen Monitor is now working. To view your child's vital charts, click "See Statistics" or choose to view current vitals from the home screen
 - 13 Check volume of linked device to ensure the App Alarm can be heard
- Your screen will flash red, an alarming message will appear, and an alarm will sound if your child's vitals fall outside of predetermined ranges.

For help with this product, please call 402.599.3339

app indicators



Indicates that your device is connected and receiving valid data.



Indicates that your device is disconnected. Either the Monitor is powered OFF, Bluetooth was interrupted, or you are outside of the 60ft connection radius.



Indicates rapid foot movement and device is waiting for deceleration of monitor to obtain accurate data.



Indicates that your device is no longer on your child's foot. Please follow instructions on page 4 to resume data readings.



App alarming function is turned ON. Alarm will sound when necessary.



App alarming function is turned OFF. Alarm will NOT sound.



Monitor battery is full



Monitor battery is depleted. App alarm will sound to alert you of depleted battery if alarming function is turned ON.



Monitor battery level is being computed

NOTE: Estimated battery life is 17 hours

For help with this product, please call 402.599.3339



care instructions

OXYGEN MONITOR

The Oxygen Monitor has a water resistant casing and the sensor head is protected with a plastic lens. Wipe with damp cloth (water only) when necessary. Ensure that sensor head is free of any residue that could block sensor from taking accurate readings.

Do not submerge in water.

Handle with care. Sensor head is fragile.

SUPPORT SOCK

The Baby Vida Support Sock can be hand-washed. Air dry.

product materials

OXYGEN MONITOR

Elastomer
RoHS compliant
BPA-Free

SUPPORT SOCK

Polyester/
Polyurethane
Fabric w/ Velcro

SUPPORT STRAP

Knit Elastic
w/ Velcro

For help with this product, please call 402.599.3339




baby vida



the mom behind the monitor

MEET MOLLIE

When Mollie and her husband were expecting their first child, they prepared in every way they knew how. They had a healthy baby boy and were overjoyed to take him home.

Mollie had an audio/video monitor to help her hear if her new baby was crying, but she started to wonder:

“What would it sound like if my baby stopped breathing?”

Through research, she found that she wasn't alone in her concern, but the right solution wasn't out there yet. So Mollie enlisted the help of renown engineers to develop this life changing device that helps bring parents vital peace of mind.

FAQs

WHAT AGES ARE THE BABY VIDA OXYGEN MONITOR FOR?

The Baby Vida Oxygen Monitor is recommended for infants ranging from one-day to one-year-old.

HOW DOES THE OXYGEN MONITOR COLLECT READINGS?

Using pulse oximetry, the Oxygen Monitor constantly monitors your baby's oxygen level and heart rate. Sophisticated algorithms were developed, giving the Monitor the capability of producing accurate outputs while taking sudden movements and normal infant behavior into consideration.

WHAT IS PULSE OXIMETRY?

Pulse oximetry utilizes the light absorptive characteristics of pulsating blood flow in the arteries to measure oxygen saturation. It is a noninvasive, painless, indicator of oxygen delivery to peripheral tissues.

WHY BLUETOOTH?

Bluetooth is the most portable, reliable, and accessible wireless connection available. Wireless internet (Wi-Fi) is unavailable in many places and can be unreliable. In addition, your App Alarm will alert you if your baby, while wearing the Monitor, is taken outside of the 60 ft connection radius. In addition, the Baby Vida App does not use cellular data to operate.

HOW LONG DOES THE BATTERY LAST?

The Oxygen Monitor's battery lasts for 17 hours and only takes 2.5 hours to fully charge.

IS IT COMFORTABLE?

Yes! Soft and flexible materials were used to create the Support Sock and Monitor Strap. The Monitor and Support Sock can be worn alone or in combination with infant socks or footed pajamas.

IS MY SMART DEVICE COMPATIBLE?

The Baby Vida App is compatible with Apple™ and Android™ phones and tablets.

DOES IT PREVENT SIDS?

No, this product does not prevent SIDS. This product is NOT a medical device. It does NOT cure, prevent or diagnose anything. It is simply a device that provides additional information for caregivers.

IS IT EASY TO USE?

Yes! The Monitor is simple to strap on your baby's foot. The Support Sock slides over the Monitor and velcro strips allow you to easily adjust to the size of your infant's leg. Then, simply download the App and follow the instructions included with your Monitor.

HOW DOES IT ALARM?

Alarming must be turned ON in your App. If your baby's oxygen level or heart rate fall outside of the predetermined range, your smart device will sound. You can use the ranges predetermined within the App or set your own using the Manual Setting.

HOW FAR CAN I BE FROM THE MONITOR?

The distance the paired device can be away from the Oxygen Monitor is approximately 60 feet. There are many factors that can affect this distance; like walls, appliances, and metal. It is important to consider the range as a bubble and make sure you walk your home to see where you are connected and where you are not prior to your first use of the Monitor. If either the Monitor or the paired device move outside of the connection radius, your App Alarm will sound.

WHAT IF THE MONITOR FALLS OFF MY BABY?

As long as the Monitor has been placed on your baby properly, it should not fall off. In the very rare case that this does happen, the App Alarm will sound to alert you.

WHAT DO I DO IF I FEEL THE PRODUCT IS NOT WORKING CORRECTLY?

For all questions, please call **402.599.3339**

warranty

Do not attempt to repair or modify this equipment. Any repairs or alterations made by any user to the equipment may void the warranty and compliance of the equipment. Changes or modifications made to this equipment not approved by BabyVida may void the FCC authorization to operate this equipment.

BabyVida guarantees this product to be free from defects in materials and workmanship under normal consumer usage for a period of 12 months from the original date of purchase, verified by sales receipt. The warranty is your exclusive warranty and is not transferable.

ANY IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL BE LIMITED TO THE DURATION OF THIS LIMITED WARRANTY, OTHERWISE THE REPAIR OR REPLACEMENT PROVIDED UNDER THIS EXPRESS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OF IMPLIED.

IN NO EVENT SHALL BABY VITA BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT OR ACCESSORY, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR OTHER FINANCIAL LOSS ARISING OUT OF OR IN CONNECTION WITH THE ABILITY OR INABILITY TO USE THE PRODUCT TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

If this product fails to function properly when used as directed under normal conditions within the 12-month period, BabyVida will either repair or replace the product, at our discretion, free of charge. This warranty does not apply to any product that has been subject to abuse, misuse, negligence or accident. Defects caused by tampering, unauthorized alterations or repairs are not covered by this warranty. We may elect to replace the entire unit, at no extra cost to you, with the same or similar model rather than repair it.

607891.2

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules and Industry Canada license-exempt RSS standards. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.



Son fonctionnement est soumis aux deux conditions suivantes: (1) cet appareil ne peut pas provoquer d'interférences, et (2) cet appareil doit accepter toute interférence, y compris celles susceptibles de provoquer le fonctionnement du dispositif.

disclaimer

Safety Instructions: WARNING

- KEEP THIS INSTRUCTION MANUAL FOR FUTURE REFERENCE.
- PLEASE READ THE WARNINGS AND INSTRUCTIONS CAREFULLY BEFORE ASSEMBLING AND USING THIS PRODUCT.
- FAILURE TO FOLLOW THE WARNINGS AND THE INSTRUCTIONS COULD RESULT IN SERIOUS INJURY OR DEATH OR DAMAGES TO THE PRODUCT.
- NOT DESIGNED FOR USE WITH ADULTS, CHILDREN, OR INFANTS OLDER THAN 12 MONTHS.

WARNING – Strangulation Hazard – Children have STRANGLED in cords. Keep the charging cord out of the reach of children. Do not put the charging cord inside a crib or play yard. Because of the serious strangulation risk to infants and toddlers, parents and caregivers should never place the charging cord within 3 feet (1 meter) of a crib. NEVER use extension cords with AC adapters. ONLY use the AC adapters provided.

- This product contains small parts. Adult assembly required. Exercise care when unpacking and assembling the product and keep small parts away from children.
- Test this monitor and all its functions so that you are familiar with it prior to actual use.
- Do not charge the baby monitor while a child is wearing it.
- Do not charge the baby monitor within 3 feet (1 meter) of a child's reach.
- Only use the charger and power adapter provided. Do not use other chargers or power adapters as this may damage the monitor.
- DO NOT use the monitor near water (such as a bath tub, sink, swimming pool, etc.).
- DO NOT use the oxygen monitor if it has been exposed to liquids.
- DO NOT immerse any part of the oxygen monitor in water or any other liquid. Clean only with a damp cloth. DO NOT spray the oxygen monitor with cleaning products or solvents.
- The support sock may be laundered on the delicate cycle or hand washed in cold water only. Air dry.
- Do not attempt to clean the oxygen monitor while it is plugged into an electrical outlet.
- Keep monitor AWAY from heat sources (such as stoves, fireplaces, radiators, etc.). Heat can damage the unit.
- Use and store the monitor at a temperature between 40°F (4°C) and 85°F (30°C).

- Do not use the monitor outdoors.
- Do not use the monitor if it has been dropped or damaged.
- Use only attachments/accessories specified by the manufacturer.
- Do not attempt to open any part of the oxygen monitor. No user-serviceable parts inside.
- Risk of electrical shock, fire or death.
- Do not attempt to remove or replace the battery. Contact Baby Vida at 402.599.3339 if you experience any problems with the battery or with charging the monitor.
- If something goes wrong, do not attempt to repair the monitor yourself. Please contact Baby Vida at 402.599.3339.

Adult Supervision

- This baby monitor is designed to provide parents and caregivers with information as an aid in the monitoring of your child. It is not a substitute for responsible and proper adult supervision.
- ALWAYS be sure to immediately check on your child when an alarm sounds.
- This monitor is not a medical device and is not intended to be used as a medical monitor. This monitor is not intended for use in the diagnosis of any disease or other conditions and is not intended to cure, mitigate, treat, or prevent any disease.
- This monitor does not prevent crib death or sudden infant death syndrome (SIDS).
- Monitoring of premature babies, or those considered to be at risk, should always be carried out under the supervision of a health professional.
- Do not leave the baby monitor in the crib or within reach of a child unless the monitor is securely strapped to the child's foot. Do not allow children to play with the baby monitor and accessories. THIS IS NOT A TOY.

IF YOU THINK YOUR INFANT MAY HAVE A MEDICAL EMERGENCY, CALL 911 IMMEDIATELY.

Customers use this monitor at their own risk. Baby Vida and its members, employees, agents, contractors, representatives, manufacturers, distributors, and associates shall not be held liable for any loss, damage, or injury suffered by the customer or any other person arising from, or in connection with, the use or manufacturer of this monitor and shall also not be held liable as the result of any defects with the monitor.



baby vida

oxygen monitor



Juvenile Products
Manufacturers
Association



www.babyvida.us • info@babyvida.us • 402.599.3339