[www.drivemedical.com](http://www.drivemedical.com)

drive  
[WWW.DRIVEMEDICAL.COM](http://WWW.DRIVEMEDICAL.COM)

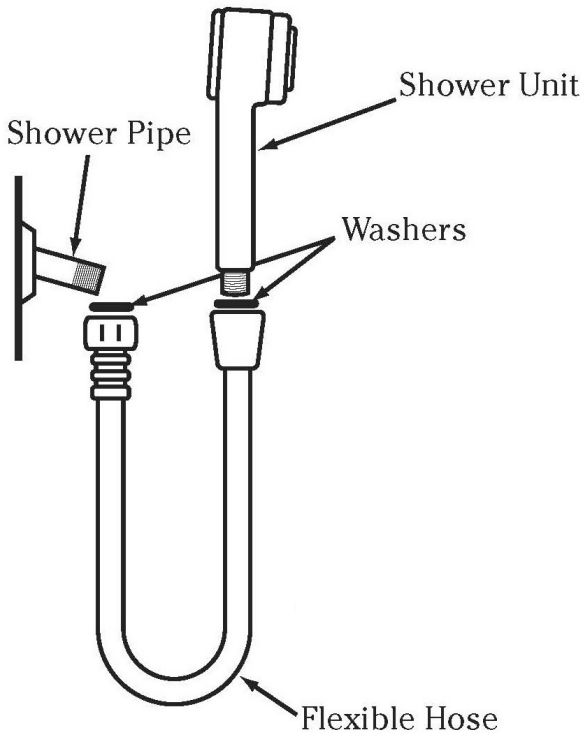
deluxe hand-held  
shower spray

rociador de ducha  
manual deluxe

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[www.drivemedical.com](http://www.drivemedical.com)

drive  
[WWW.DRIVEMEDICAL.COM](http://WWW.DRIVEMEDICAL.COM)



## to install as shower head

1. Remove the old shower head leaving shower arm.
2. Screw diverter valve into existing shower head.
3. Place rubber gasket (included) into diverter valve.
4. Screw deluxe shower spray head into diverter valve.

## to install as hand-held shower spray

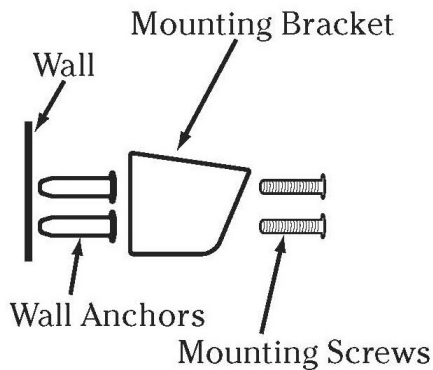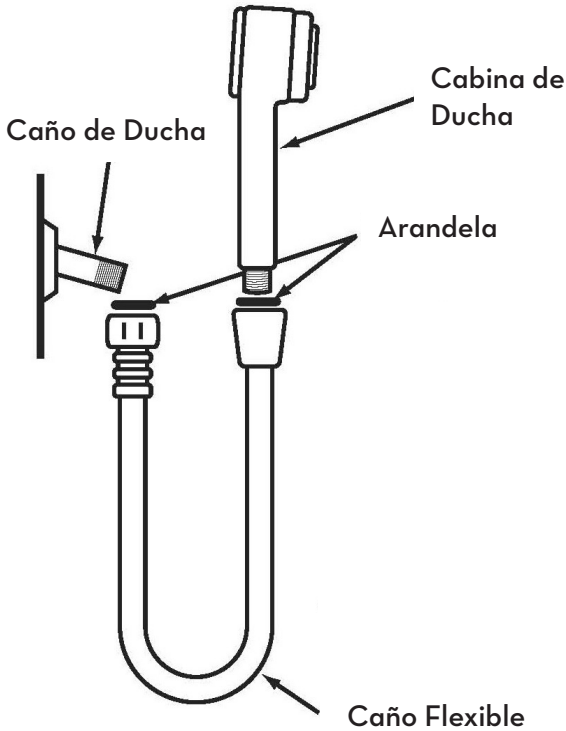1. Remove the old shower head leaving shower arm.
2. Screw diverter valve into existing shower arm.
3. Place rubber gasket (included) into diverter valve.
4. Attach shower hose to diverter valve
5. Screw shower head onto shower hose
6. Mount bracket onto wall with screws (included).
7. Hang shower head on bracket when not in use.

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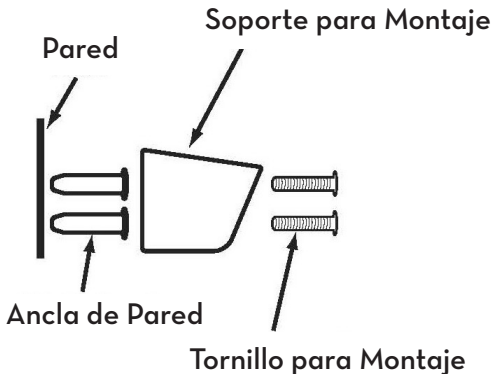
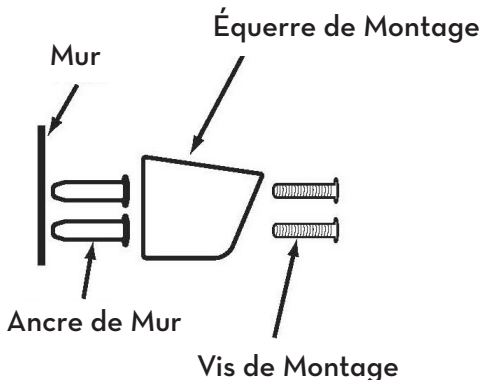
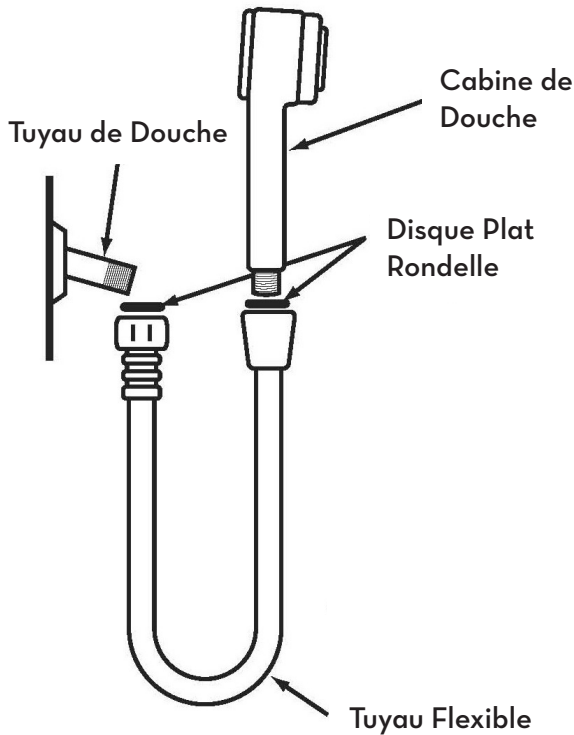
## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in Taiwan

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.